**DDNature<sup>ZR</sup>**

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Instructions  
for use

CLOSE TO YOU

 **Dental  
Direkt**

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## If you expect more than standard, but want a fast and secure way to reach your goal.

We live up to our own demands for brilliant, aesthetic dental prostheses with the innovative DD Nature Zr porcelain veneers.

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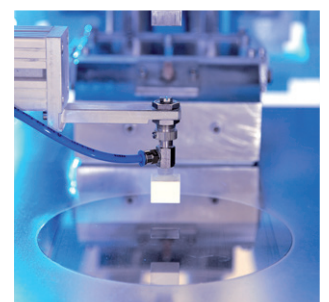
The name is a commitment – almost identical to the natural model.

The concept allows you to achieve lively, reliable colour renditions with the least effort. A veneer system that adapts to your laboratory routine and not vice-versa.

As a ceramic specialist and one of the leading zirconia producers, we ensure that all processes are coordinated with each other, from the raw material to the finished milling blank and ceramic powder. We provide you with the basis for strong, flawless bonding practically ex works.

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DD Nature ZR is a veneering ceramic that is optically and technically perfectly attuned to zirconia frameworks (Y-TZP). TEC has been optimised for very strong bonding with zirconia (TEC  $10.5 \cdot 10^{-6} \text{K}^{-1}$ ).

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DD Nature ZR is unsuitable for veneering metal or titanium frameworks

DD Nature ZR must not be used in case of intolerance to the materials used

DD Nature ZR is solely intended for dental use.

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Avoid breathing in milling dust

Observe the latest version of the safety data sheet

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
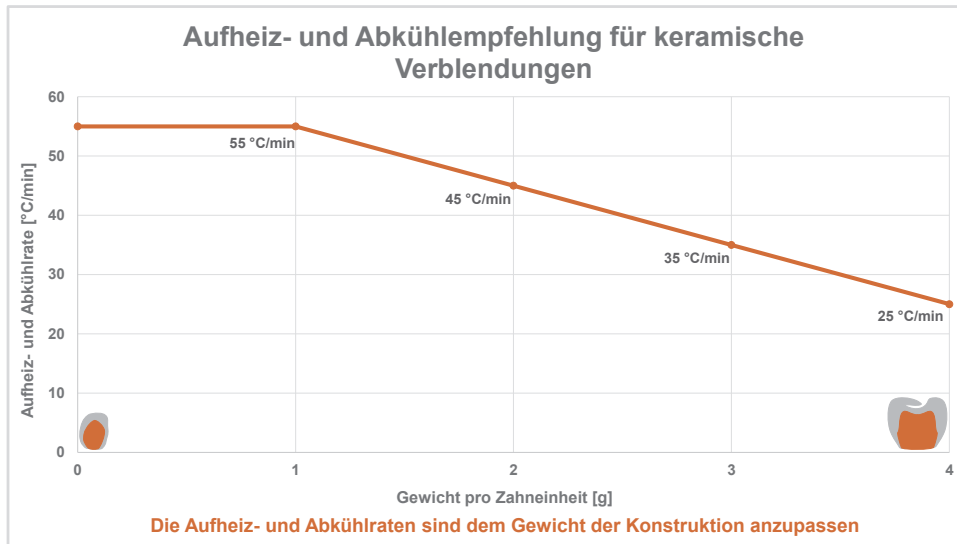
The firing parameters given above are guidelines, which must always be adjusted to suit the furnace used and the situation of the furnace. What is essential is getting the right firing result. Best way is to fire a test body made of T-clear and compare the transparency of the supplied indicator.

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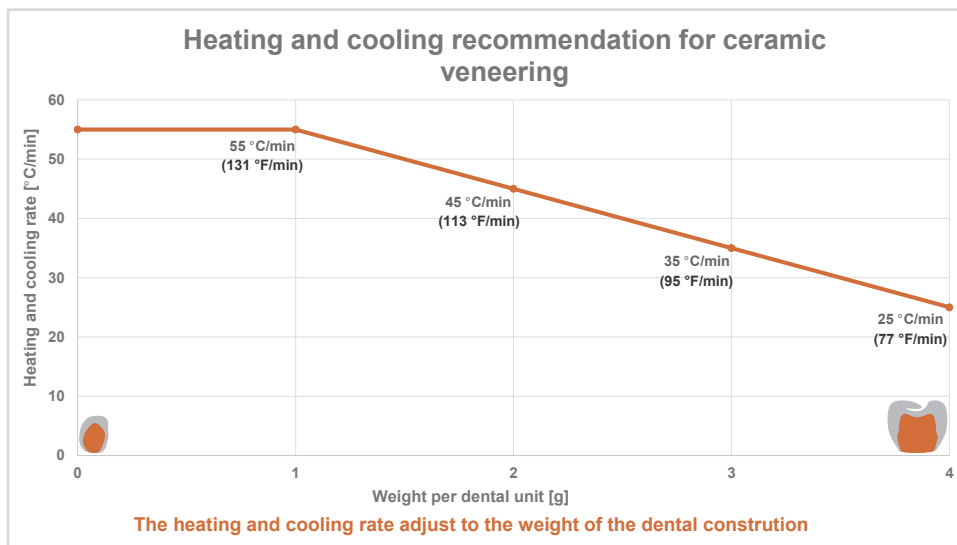
Minimize the number of firings and adjust heating and cooling rates to material and construction dimension to avoid thermal stress.

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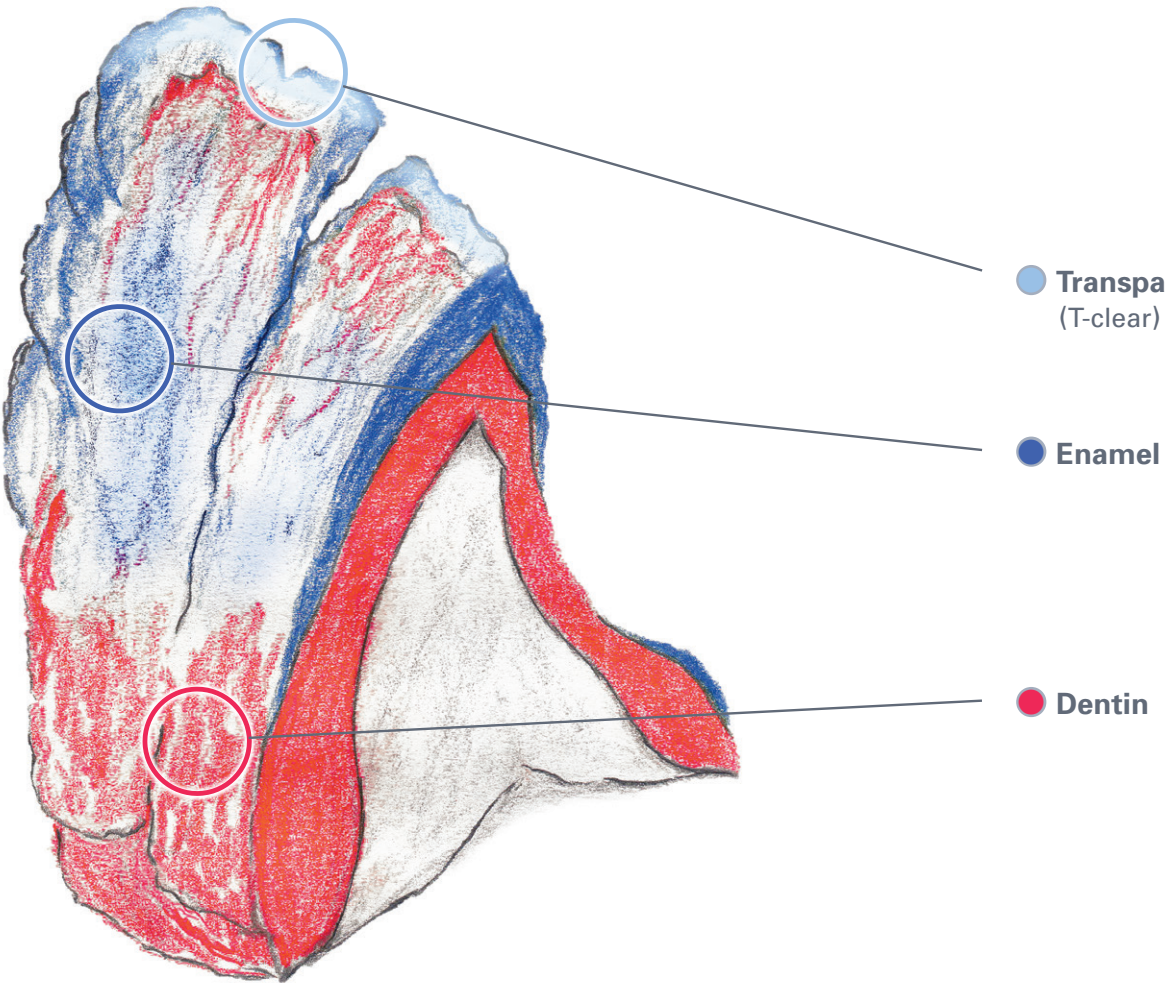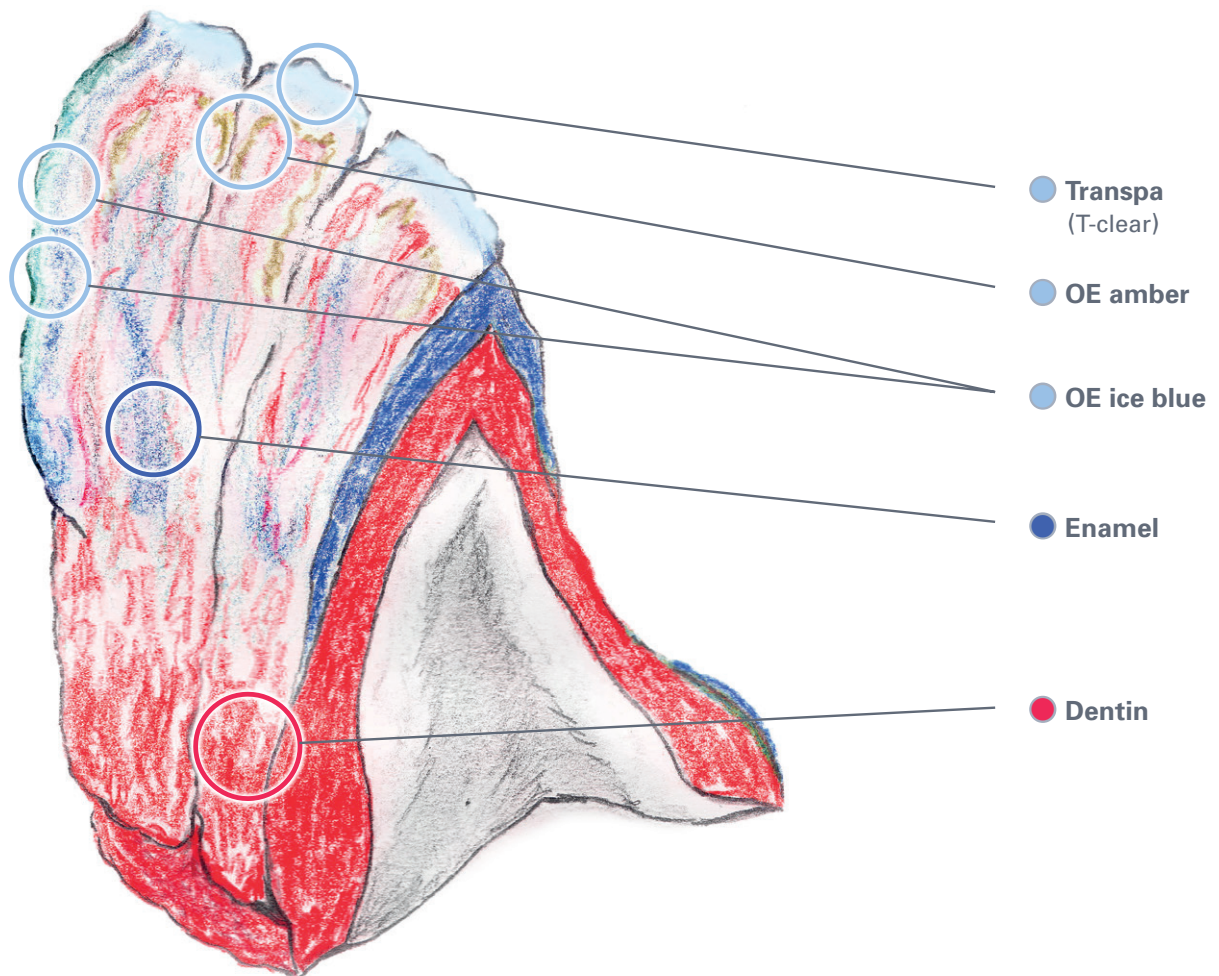
The labial and occlusal form is first built up with dentine material in the required shade. Once the anatomical form is ready, reduce the incisal and interdental areas of the dentine in order to create enough space for further layering.

Then complete the missing areas with enamel and transparent porcelains or effect transpa/transpa opal porcelains – depending on the individual appearance of the crown required.

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Note: For the wash firing, a thin layer of dentin mass (no correction mass!) should be applied to the entire framework. Due to the higher temperature and the thin ceramic layer compared to all subsequent fires, the wettability of the zirconium framework is increased and the bond to the layered ceramic is improved.

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The correction firing serves to optimise the form and supplement the fire shrinkage.

Before layering, the crown must be prepared and cleaned.

The glaze firing occurs after preparation, surface contouring and of course final cleaning.

The stains from Dental Direkt provide the perfect finish for veneered and monolithic constructions.

# DDNature<sup>ZR</sup>

CE 0482

Dental Direkt GmbH  
Industriezentrum 106 – 108 | 32139 Spenge | Tel: +49 5225 86319-0  
E-Mail: [info@dentaldirekt.de](mailto:info@dentaldirekt.de) | [www.dentaldirekt.de](http://www.dentaldirekt.de)

Rev. 07\_2021/09